* nosiboo[®]

nasal aspirator

User manual **ENU**
Nosiboo Nasal Aspirator

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Manufactured for
ATTRACT Kft. Pécs, Hungary
MADE IN HUNGARY

2016-12-01
DMR-02/A04/V01
E-Mail: customerservice@nosiboo.com
Web: www.nosiboo.com
Customer service: 1-312-957-6046

Warning and safety symbols – You may see the following symbols in the user manual or on the label of the device:

Name and address of the manufacturer

ENU

Read the user manual

Warning

Caution

REF

Reference number

LOT

Lot number

Important safety regulations

Use only as directed. Misuse can cause injury or death.
Do not allow children to play with the device as it is NOT A TOY.
Do not leave the parts of the dismantled Colibri head within reach of children. Children may swallow the small parts.

The device is intended to be used only for removing nasal mucus from a child's nose.
Please consult your healthcare provider about when and how often you should clear your baby's nose and if redness occurs.
Nose drops and other preparations should be used as directed by your healthcare professional.
Keep out of reach of children.
Make sure that all parts are properly attached before using the device.

Always wash the Colibri head after use.

Read and follow the instructions of the “Environment” section if you would like to dispose of the device or any part of it.

Do not insert the nose tip into the baby’s mouth or ears.

Do not force or insert the nose tip too far into the nose.

Do not press the tip against the side of the nostril.

Do not use if nasal sores, inflammation or nosebleed are present. If pain or bleeding occurs, stop use immediately and consult your doctor.

Do not use the Colibri head and the device without the collared tube.

Make sure that all parts are dried thoroughly before the next use.

Do not use more than a teaspoon of saline solution at a time.

Keep away from heat.

1. Intended use

For aspirating nasal mucus from a child’s nose. The vacuum needed for the aspiration is created by the user’s mouth (in an indirect way thanks to the detached design).

Intended purpose: removing nasal mucus from the baby’s nose to prevent complications caused by childhood colds and to accelerate the healing process.

Recommended for: relieving symptoms of the common cold.

2. Device introduction and assembly

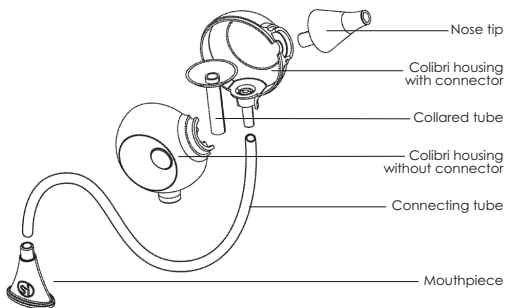
The aspirated liquid accumulates on the inner wall of the Colibri head. Thanks to its exclusive patented design, the liquid cannot exit from the inside of the Colibri head, safely preventing the nasal mucus from getting into the connecting tube or into the mouth of the person performing the suction.

There is no need for a filter!

The Nosiboo Nasal Aspirator can hold up to a teaspoon of thin liquid.

Assembly

1. Clean the parts of the Colibri head (see chapters “Cleaning” and “Maintenance and storage”).
2. Insert the collared tube into one half (the half with connector) of the Colibri head.
3. Put the other half of the Colibri head (without connector) on the first half.
4. Push the flexible nose tip on the matching flange.
5. Attach one end of the connecting tube to the Colibri head connection point.
6. Attach the other end of the connecting tube to the mouthpiece.

3. Instructions for use

1. Infants should be reclining or in their parent's arms; small children should be sitting in a stable position while their nose is being aspirated.
2. Put the mouthpiece into your mouth and gently insert the thoroughly cleaned nose tip into the child's nasal cavity.
3. Make sure that the connection point of the connecting tube and the Colibri head points downwards.
4. Slightly move the nose tip around while intermittently performing suction with your mouth through the connecting tube. Do not force or insert the nose tip too far into the nostril.
5. Disassemble and clean the Colibri head according to the instructions in the "Cleaning" section.

TIP: Move the nose tip around to make sure that you get all areas of the nostril.

TIP: Dry or very thick nasal mucus can be dissolved by a mild saline solution. No more than a teaspoon of these liquids should be used at a time. Wait a few minutes after using the solution, wipe excess discharge with a tissue, and then use the nasal aspirator.

4. Cleaning

Cleaning the device is very simple.

1. Disconnect the connecting tube and the nose tip from the Colibri head.

2. Take the Colibri head apart.

3. Disconnect the mouthpiece from the connecting tube.

4. Wipe the connecting tube with a dry cloth.
(If used according to the instructions, the connecting tube does not need washing.)

5. Wash all the other parts in lukewarm water and dishwashing detergent and **let them dry**.

5. Maintenance and storage

The device should be stored in a dry place protected from dust and damage (e.g. in a cabinet).

If you have multiple children, we recommend using a separate nasal aspirator for each child.

6. Environment

The Nosiboo Nasal Aspirator does not pose any environmental risks and can be disposed of with household waste.

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