*[®]nosiboo[®]pro

electric nasal aspirator

User manual **ENG**
Nosiboo Pro Nasal Aspirator

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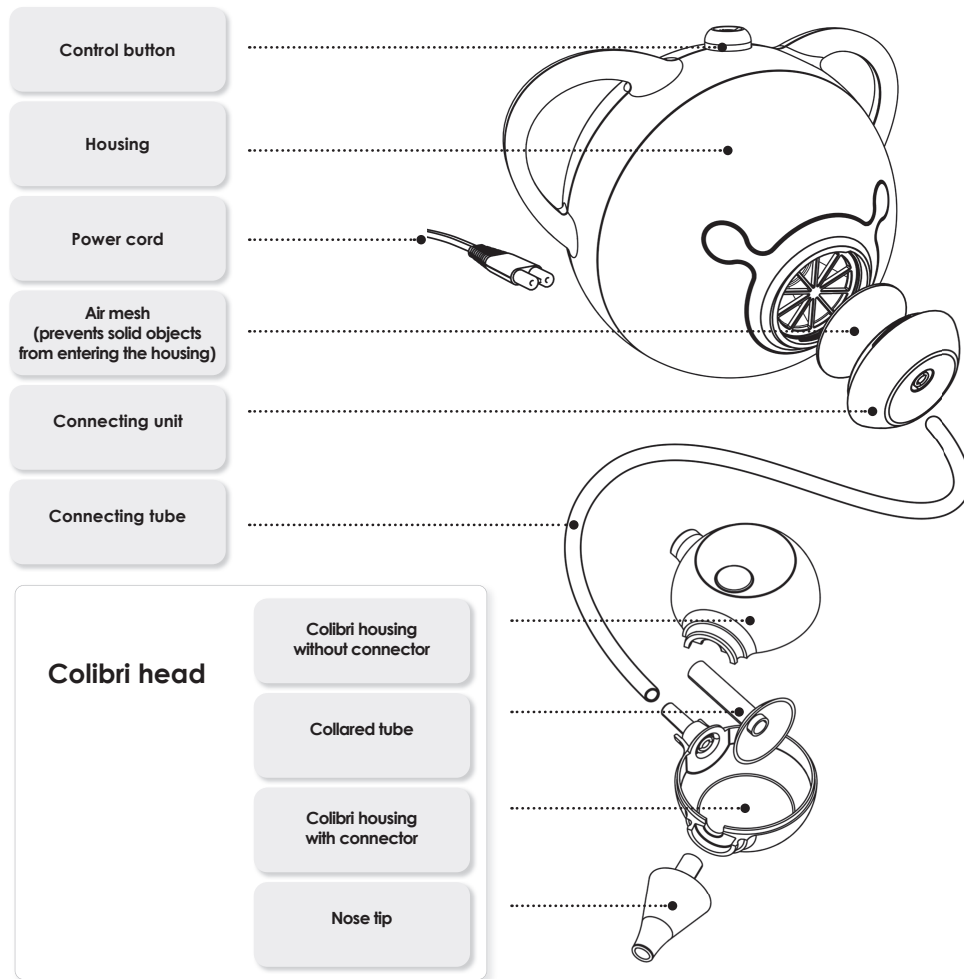
USER MANUAL

Foreword

Dear Customer,
Thank you for purchasing our KSCL, PTPA, Red Dot Design and German Design award-winning Nosiboo Pro Nasal Aspirator.

Please follow the instructions provided in this user manual to obtain the very best performance from your nasal aspirator.

Table of contents

1. Package contents
2. Safety instructions
3. Intended use
4. Before use
5. Use
6. Cleaning
7. Storage and maintenance
8. Troubleshooting
9. Important information about electromagnetic interference
10. Environment
11. Specifications
12. Accessories, spare parts

Read all instructions before use.

Nosiboo Pro Nasal Aspirator is a medical device. You may only return the device for a refund if the device is unused and undamaged and the protective foil on the Colibri head and on the device are intact.

1. Package contents

Nosiboo Pro

Connecting tube

Colibri head with nose tip

Nose tip

Power cord

Attract Kft. is an ISO 13485 certified medical device manufacturer. This product is an MDD Legacy Device.

Actor EUDAMED No.: HU-MF-000004252

MANUFACTURED BY ATTRACT KFT. Pécs, Hungary
MADE IN HUNGARY

E-Mail: customerservice@nosiboo.com
Customer service: 1-312-957-6046

Website: www.nosiboo.com

DMR-01/A05/V09
2022-08-07

Page: 3/50

2. Safety instructions

Safety signs table

Type BF applied parts
(increased protection against
electric shock)

Double insulation

Class IIa medical device

Name and address of the
manufacturer

Read the user manual!

For home use only

Item number (type number)

Serial number (Unique device identifier)

Global Trade Item Number (UDI-DI)

Lot number (production identifier)

The housing protects against solid objects
over 12.5 mm (4.92 inches) entering the
device; however, it does not protect
against the infiltration of liquids.

Keep dry!

Warning

Caution

DO

DON'T

Hazardous waste

General use

Use only as directed. Misuse can cause injury
or death.

The device should only be used by adults.

Do not allow children to play with the device,
it is **NOT A TOY**. (The power cord can easily
wind around a child's neck.)

Do not leave the device unattended near children.

Do not use the device in the bathroom or in any
other humid or wet environment.

Do not handle the power cord with wet hands.

Do not cover the vents on the back of the
device while in use. Hot air exits the device
through the vents.

Do not use the nasal aspirator for nasal washing!

The device is intended to be used only for
removing nasal mucus from a child's nose.

Please consult your healthcare provider about
when and how often you should clear your
baby's nose or if redness occurs.

Keep out of reach of children.

Make sure that all parts are properly attached
before using the device.

Make sure that the housing, the connecting
tube and the Colibri head are not damaged
or cracked.

Double-check that you have inserted the
collared tube into the Colibri head after
cleaning.

Use the device only with the connecting tube and Colibri head supplied by the manufacturer.

Only use the device when the connecting tube is dry.

While in use, the device should be in a stable position, standing on a flat surface.

The connection point of the Colibri head and the connecting tube should point downwards while in use.

Stop using the device if mucus or saline solution has been drawn into the connecting tube. Disconnect it from the electrical outlet, empty the Colibri head, clean and dry the Colibri head and the connecting tube, check the air mesh and replace it if it is wet or clogged.

Stop using the device if liquid or solid has been drawn into the housing. Contact our customer service.

Always switch off the device after use by pushing the control button.

Always thoroughly clean the Colibri head after use.

Read and follow the instructions in the "Important information about electromagnetic interference" section.

Read and follow the instructions in the "Environment" section if you would like to dispose of the device or any part of it.

Do not insert the nose tip into the baby's mouth or ears.

Do not force or insert the nose tip too far into the nose.

Do not press the nose tip against the side of the nostril.

Do not use if nasal sores, inflammation or nosebleed are present. If pain or bleeding occurs, stop use immediately and consult your doctor.

Do not use the device if you see that the connecting tube or the Colibri head are damaged. Replace the damaged part. (A list of spare parts can be found in the "Accessories, spare parts" section.)

Do not use the Colibri head and the device without the collared tube.

Do not use more than one teaspoon of saline solution. Excess solution can enter into the connecting tube and can cause damage to the device.

The device should not be used in a clinical environment.

Do not attach the Colibri head to other devices. Do not unplug the device from the electrical outlet until it is completely shut down.

Cleaning, Maintenance and Storage

Protect the power cord against damage.

Keep the power cord away from heat (e.g. furnace, heater or convector) because it can cause damage to the insulation.

Using a damaged power cord can cause a short circuit (short-circuit fire).

When assembling, disassembling, and cleaning the device, turn it off and unplug the unit from the electrical outlet by grasping the plug.

Do not break the power cord, do not place heavy items on the cord, to avoid damage. A damaged cord may cause electric shock.

A broken, damaged, worn, frayed or short-circuited power cord can cause an electrical fire, so do not use it!

Do not remove the mounting screws from the housing. Do not reconstruct the device for other purposes.

Do not leave the parts of the dismantled Colibri head within reach of children. Children may swallow the small parts.

Do not wash the housing or the power cord in water or any other liquid. Do not handle the power cord with wet hands. This may cause electric shock or malfunction.

Make sure that all parts are dried thoroughly before the next use.

Double-check that you have inserted the collared tube into the Colibri head after cleaning.

Always unplug the device from the outlet when not in use.

Unplug the power cord from the mains socket only by grasping the plug.

To prevent damage, the connecting unit should not be exposed to heat.

Store in a cool, dry place. Avoid direct sunlight or heat.

Power Source / Electrical Power

Protect the power cord against damage.

Switch off the device and unplug the unit from the electrical outlet when assembling, disassembling and cleaning the device.

In case of unexpected failure or malfunction, immediately switch off the device and unplug it from the electrical outlet.

Be sure to use the device only with a 110V AC.

Do not use the device if there is a breakage or crack in the housing or power cord.

Do not break the power cord, and do not place heavy items on it, because a damaged cord may cause electric shock.

When not in use, always unplug the power cord from the outlet by grasping the plug.

Do not unplug the device from the electrical outlet until it cycles off completely.

3. Use

The Nosiboo Pro Nasal Aspirator is intended for removing nasal mucus from a child's nose.

The Colibri head has a patented design that prevents mucus from entering the device and makes it easier to clean the Colibri head, simplifying the daily use of the nasal aspirator.

Intended purpose:

- * to ensure clearer nasal breathing for infants
- * to relieve symptoms of common childhood colds by aspirating nasal mucus
- * to relieve childhood allergy-like symptoms by removing nasal mucus

Do not use:

- * if the child has hemophilia
- * if the child has frequent nosebleeds
- * if the child is taking blood-thinning medicines
- * in the event of congenital malformations (e.g. cleft palate)

The device should only be used by adults.

The device is intended to be used only for removing nasal mucus from a child's nose.

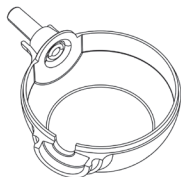
The device is not a nasal irrigator! Do not use large amounts of liquid during nasal aspiration!

4. Before use

Disassemble the Colibri head, then clean and dry its parts.

The parts of the Colibri head:

Colibri housing with connector

Colibri housing without connector

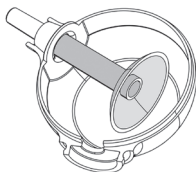
nose tip


collared tube


1. Assemble the Colibri head.

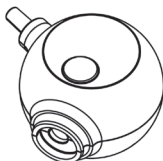
a) Insert the collared tube into the side of the Colibri housing that is supplied with a connector.

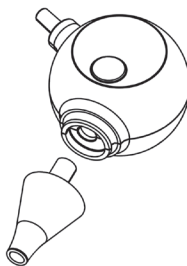b) Place the Colibri housing that has no connector in the correct position.

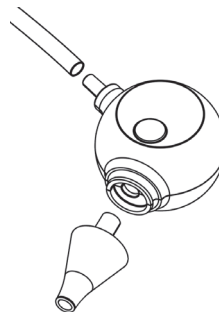
c) Push the nose tip into place.


d) Attach one end of the connecting tube to the Colibri head.

2. Attach the Colibri head to the device.

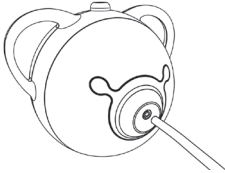
- a) Attach the free end of the connecting tube to the device housing.

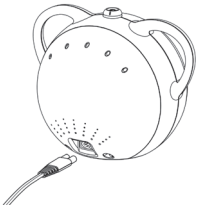- b) Attach one end of the power cord to the device housing.

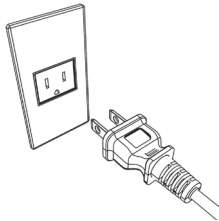3. Plug the other end of the power cord into a 110V AC outlet.

4. The device is ready for use.

Be sure to use the device only with a 110V AC.
Failure to do so may cause electric shock or fire.

Clean the parts of the Colibri head before every use.

Make sure that all parts are cleaned and dried thoroughly before use.

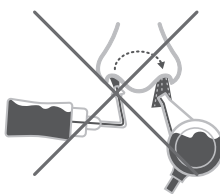
Do not use the Colibri head and device without the collared tube.

5. Instructions for use


Do not aspirate water with the device!

Do not use the nasal aspirator for nasal washing!

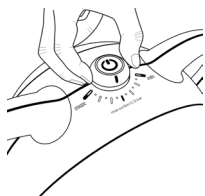1. Select the correct suction power.

low setting for newborns

medium setting for babies

high setting for small children and for preschoolers

TIP: If the mucus is thin, you can use the device at a lower setting. If it is thick you can use the device at a higher setting than normal.

2. Hold your child in a stable position.

Sit your child onto your lap, hold them in your arms or sit them into an upright position supporting their back with a pillow. You may also sit them in an infant bouncer or infant seat.

3. Insert the clean, dry nose tip into the child's nostril.

The connection point of the connecting tube should point downwards while in use.

NOTE: Do not pinch or block the other nostril while using the nasal aspirator.

In rare instances, it is possible that the nostrils of a newborn are too small for inserting the nose tip. In this case, place the nose tip in a way that the nostril is completely covered. The nasal aspirator's efficiency is enough for cleaning the nasal passages properly.

4. Turn on the device by pushing the control button.

NOTE: Any time you are not actively using the device, switch it off. After the motor cycles off completely, you can restart it.

The device turns itself off automatically after 13 minutes of continuous use but it can be restarted. In case of overuse or if insufficient ventilation is provided, the thermal switch switches off the device and will prevent reuse for approx. 45 minutes for safety reasons.

5. Move the nose tip around gently to make sure that you get all areas of the nostril.

The suggested length of contact aspiration is 10 seconds per nostril.

Dry, very thick mucus can be dissolved by a mild saline solution. After using the solution, wait a few minutes, wipe excess discharge with a tissue, then reuse the nasal aspirator as directed.

To avoid damage to the device, do not use more than a teaspoon of solution at a time. The patented design of the Colibri head can hold up to a teaspoon of thin liquid and prevent mucus from entering the connecting tube.

The device is not a nasal irrigator!

The device should only be used by adults.

Do not allow children to play with the device as it is NOT A TOY.

Do not cover the vents on the back of the device while in use. Hot air may exit the device through the vents.

Do not use the device in the bathroom or in any other humid or wet environment.

Do not handle the power cord with wet hands.

Do not break the power cord during use or storage.

6. Switch off the device after use with the control button.

NOTE: Always switch off the device after use.

The motor will run for a short time to cool down after you have switched off the device.

TIP: The Nosiboo Pro Nasal Aspirator can be used several times a day. It is recommended that you clear the airways before bed time and feeding to ensure the child breathes peacefully.

NOTE: Nose drops and other preparations should be used as directed by your healthcare professional.

Make sure that all parts are dried thoroughly before use.

Always switch off the device after use with the control button.

The device must be used on a flat, hard surface!

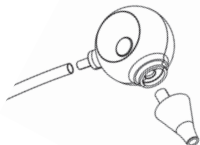
Do not unplug the power cord from the electrical outlet until the device is completely shut down.

Do not use the Colibri head and the device without the collared tube.


Do not cover the device, and do not use it on a soft surface which could block the ventilation holes (couch, pillow, bedclothes etc.)

6. Cleaning


1. Remove the connecting tube and the nose tip, then take apart the Colibri housing.

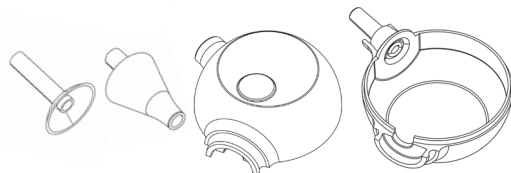
2. Wash the collared tube, the nose tip, and the Colibri housing in lukewarm water and dishwashing detergent and let them dry.

Use the brush to clean the nose tip.

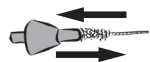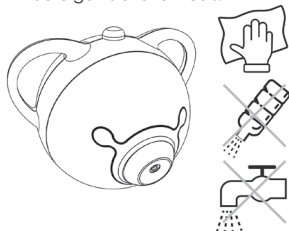3. Clean the housing with a dry, soft cloth; do not use any liquids, detergents or chemicals.

4. Wipe the connecting tube with a cloth if necessary.

Switch off the device and unplug the unit from the electrical outlet when assembling, disassembling and cleaning its parts.

Do not wash the housing or the power cord in water or any other liquid.

Do not handle the power cord with wet hands.

Do not wash the connecting tube.

Thoroughly clean the Colibri head after use.

Make sure that all parts are dried thoroughly before the next use.

If liquid gets into the connecting tube due to improper use, make sure that it is completely dry before using it again.

Do not disassemble the housing.

To avoid damage, make sure the device is not exposed to heat.

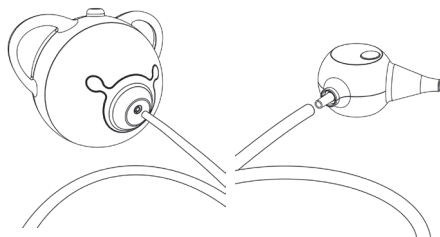
7. Storage and maintenance

Storage

1. Unplug the device after it has completely shut off and the cooling phase is over.

NOTE: After the cooling process has stopped, the power cord should be unplugged from the electrical outlet and from the device.

2. Disconnect the connecting tube from the device and the Colibri head.

3. Keep the device in an enclosed space where it is protected from dust and damage (e.g. in a cabinet).

NOTE: Do not store near a heat source.

Maintenance

There is a special air mesh behind the connecting unit which prevents solid objects and dust from entering the housing. In the event of an unexpected drop in performance, it is recommended that you check the air mesh before the end of the cycle. If the air mesh is dirty or clogged, replace it with a spare air mesh. (A list of spare parts can be found in the "Accessories, spare parts" section.)

Protect the power cord against damage.

Keep the power cord away from heat.

Do not leave the parts of the dismantled Colibri head within the reach of children.

Do not break the power cord, and do not place heavy items on it, because a damaged cord may cause electric shock.

Do not use any liquids, detergents, or chemicals for cleaning the device housing and the connecting tube.

Do not wrap the power cord around the device.

A damaged power cord can cause electrical fire!

Make sure that all parts are dried thoroughly before assembly.

Store the device in a clean, dry place. Avoid direct sunlight or heat.

8. Troubleshooting

Malfunction	Reason	Recommendation
The device cannot be switched on.	The plug is not connected.	Connect the power cord to the device, then to the 110 V socket.
The device cannot be switched on.	The device automatically switched off for safety reasons due to overuse.	For the sake of your child's health, stop using the nasal aspirator. The device can be used again in approx. 45 minutes.
The suction intensity is decreased.	The nose tip is blocked or the Colibri head was incorrectly assembled.	Take the Colibri head apart and clean it. After drying the parts of the Colibri head, assemble them according to the user manual.
The suction intensity is decreased.	The air mesh is clogged.	Replace the air mesh according to the user manual's "Cleaning, maintenance and storage" section.
Liquid or mucus got into the connecting tube.	The Colibri head was not held in the correct position.	Switch off the device and clean and dry the Colibri head and the connecting tube. Check the air mesh and replace it if necessary.
Liquid or mucus got into the connecting tube.	The connection point of the tube is not pointed downwards.	Switch off the device and clean and dry the Colibri head and the connecting tube. Check the air mesh and replace it if necessary.
Liquid or mucus got into the connecting tube.	Too much saline solution was used.	Reduce the volume of the saline solution to less than a teaspoon (0.17 oz, 5 mL). Switch off the device and clean it, then dry the Colibri head and the connecting tube. Check the air mesh and replace it if necessary.

9. Important information about electromagnetic interferences

Home wireless network equipment, mobile phones and wireless phones may affect the operation of certain medical equipment, as electromagnetic interference can occur between devices. Electromagnetic interference may cause harmful interference in the operation of certain medical devices, potentially causing emergency situations.

Due to the above mentioned reasons, medical devices must not interfere with any other equipment.
Do not use near high-performance radiation-emitting devices.

The Nosiboo Pro Nasal Aspirator is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled.

The Nosiboo Pro Nasal Aspirator has been tested and found to comply with the requirements of EN 60601-1-2:2007 and of FCC Part 15, 2007.

Disturbance emission tests

Description of the tests	Limits and test levels of the related Standard	Result
Radiated RF emission test	FCC Part 15, 2007 30-3000 MHz class B	Passed
Conducted disturbance voltage on AC	FCC Part 15, 2007 0.15-30 MHz class B	Passed
Flicker	EN 60601-1-2:2014 EN 61000-3-3:2009	NA

Immunity tests

Description of the tests	Limits and test levels of the related Standard	Result
Immunity against radiated RF disturbances	EN 60601-1-2:2014 3V/m 0.08-2.5GHz Modulation: 1 kHz, 80 % AM	Passed Evaluation: operation in compliance with the specification
Immunity against conducted RF disturbances on AC power and patient cables	EN 60601-1-2:2014 3 Veff 0.15 – 80 MHz Modulation: 1 kHz, 80 % AM	Passed Evaluation: operation in compliance with the specification
Immunity against EFT signals	EN 60601-1-2:2014 ±2 kV between L, N and ref. ground	Passed Evaluation: operation in compliance with the specification
Immunity against surge voltages	EN 60601-1-2:2014 ±1 kV between L and N	Passed Evaluation: operation in compliance with the specification
Immunity against voltage dips	EN 60601-1-2:2014 100 %, 60 %, 30 % 0.5, 5, 25 periods	Passed Evaluation: operation in compliance with the specification
Immunity against short voltage interruptions	EN 60601-1-2:2014 100 % for 5 periods	Passed Evaluation: operation in compliance with the specification
Immunity against electrostatic discharges (ESD)	EN 60601-1-2:2014 ±8 kV air, ±6 kV contact	Passed Evaluation: operation in compliance with the specification

Flicker measurement was omitted because no flicker was observed.

10. Environment

Dispose of the device in accordance with your local recycling and waste disposal practices and requirements.

The Colibri head does not present a risk to the environment and can be disposed of as household waste.

11. Specifications

Adjustable motor performance: 210–380 W

Differential pressure: 0.03–0.08 bar

Max. air flow:

17,44 gpm (66 L/min) measured on the Nosiboo Pro housing

7,93 gpm (30 L/min) measured on the Colibri head

Power supply: 110V / 60Hz

IP class: IP21

Protection class: II

Type of applied parts: BF

Applied parts: the complete housing

Operation type: no continuous operation

Operation length: on 13 min/approx. off 45 min

Duration of continuous use: max. 13 min

Tripping temperature of the thermal switch: 203°F (95°C)

Average housing temperature: 86–95°F (30–35°C) after 1 min of continuous use

Noise level: max. 70 dB

Operating temperature range: -10 °C... +40 °C

Operating relative humidity: max. 75%

Storage and transportation temperature: -20 °C...+65 °C

Storage and transportation relative humidity: 10...95%

The expected life of the housing (electrical parts): 3 years

Expected life of the Colibri head: 3 months

Diameter of the nasal insertion end of the Colibri head:

0,22 Inches (5,6mm)

Length of the nasal insertion end of the Colibri head:

min. 1,26 Inches (32mm)

Weight (housing): 2.26 lbs (1027g)

Dimensions (housing, W x H x L): 7.874 x 6.102 x 6.890 Inches
(200 x 155 x 175 mm)

Dimensions (complete package): 7.087 x 7.087 x 8.858 Inches
(180 x 180 x 225 mm)

We reserve the right to make technical changes.

12. Accessories, spare parts

Nosiboo Pro Accessory Set:

Accessory pack consisting of Nosiboo Pro spare parts
www.amazon.com

Parts can be ordered from the customer service:

Phone: 1-312-957-6046

E-mail: customerservice@nosiboo.com

Power cord
NO-01-022-A6-0WU1

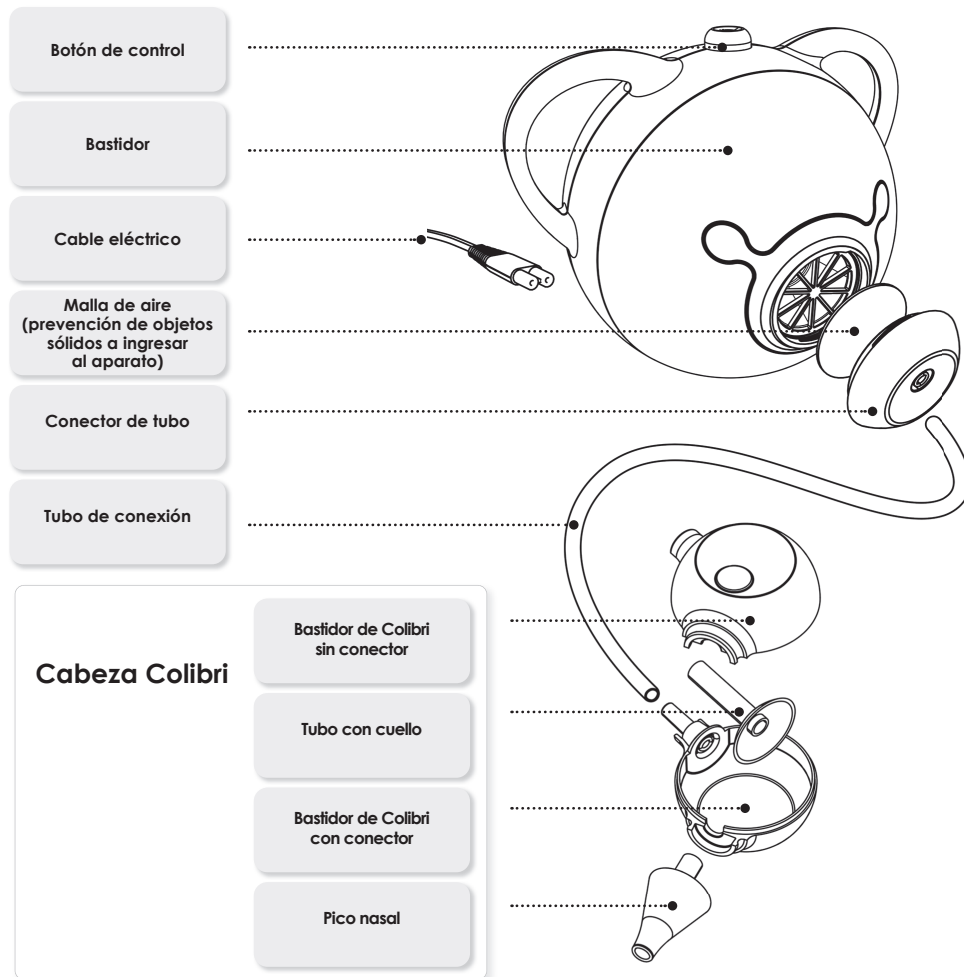
Connecting unit
NO-01-050-H2-7BXX * blue
NO-01-050-H2-7PXX * pink
NO-01-050-H2-7GXX * green

The manufacturer shall not be held responsible for the failure of products from unauthorised import sources!

For more information on the warranty, please see the link below:

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