# nosiboo<sup>\*</sup>pro<sup>®</sup>

electric nasal aspirator

## User manual

Nosiboo Pro Nasal Aspirator

ENU

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Aspirador Nasal Nosiboo Pro

ESP

## Mode d'emploi

Aspirateur Nasal Nosiboo Pro

FRC

**BPA  
FREE**


reddot award 2014  
winner

GERMAN  
DESIGN  
AWARD  
SPECIAL  
2016

nosiboo<sup>\*</sup>pro<sup>®</sup>  
electric nasal aspirator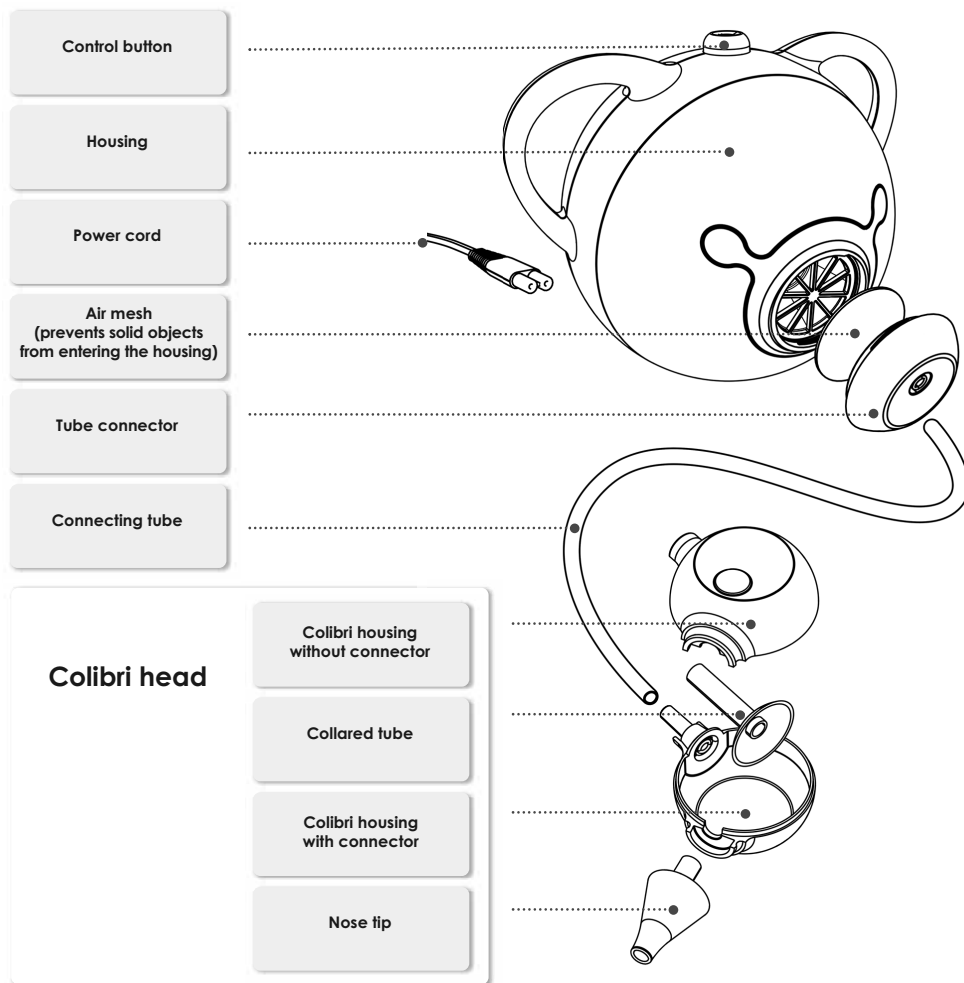

# USER MANUAL

## Foreword

Dear Customer,

Thank you for purchasing our Red Dot Design and German Design award-winning Nosiboo Pro Nasal Aspirator.

Please follow the instructions provided in this user manual to obtain the very best performance from your nasal aspirator.

Please keep this manual for warranty purposes and future reference.

## Table of contents

1. Package contents
2. Safety instructions
3. Intended use
4. Before use
5. Instructions for use
6. Cleaning
7. Storage and maintenance
8. Troubleshooting
9. Important information about electromagnetic interference
10. Environment
11. Specifications
12. Accessories, spare parts
13. Warranty

**Read all instructions before use.**

**Nosiboo Pro Nasal Aspirator is a medical device.** You may only return the device for a refund if the device is unused and undamaged and the protective foil on the Colibri head and on the device are intact.

## 1. Package contents

Nosiboo Pro

Connecting tube

Colibri head

Nose tip

Power cord

ENU

Use the device only with the connecting tube and Colibri head supplied by the manufacturer.

Only use the device when the connecting tube is dry.

While in use, the device should be in a stable position, standing on a flat surface.

The connection point of the Colibri head and the connecting tube should point downwards while in use.

Stop using the device if mucus or saline solution has been drawn into the connecting tube. Disconnect it from the electrical outlet, empty the Colibri head, clean and dry the Colibri head and the connecting tube, check the air mesh and replace it if it is wet or clogged.

Stop using the device if liquid or solid has been drawn into the housing. Contact our customer service.

Always switch off the device after use by pushing the control button.

Always thoroughly clean the Colibri head after use.

Read and follow the instructions in the "Important information about electromagnetic interference" section.

Read and follow the instructions in the "Environment" section if you would like to dispose of the device or any part of it.

Do not insert the nose tip into the baby's mouth or ears.

Do not force or insert the nose tip too far into the nose.

Do not press the nose tip against the side of the nostril.

Do not use if nasal sores, inflammation or nosebleed are present. If pain or bleeding occurs, stop use immediately and consult your doctor.

Do not use the device if you see that the connecting tube or the Colibri head are damaged. Replace the damaged part. (A list of spare parts can be found in the "Accessories, spare parts" section.)

Do not use the Colibri head and the device without the collared tube.

Do not use more than one teaspoon of saline solution. Excess solution can enter into the connecting tube and can cause damage to the device.

The device should not be used in a clinical environment.

Do not attach the Colibri head to other devices.

Do not unplug the device from the electrical outlet until it is completely shut down.

## 2. Safety instructions

### Safety signs table

Type BF applied parts (increased protection against electric shock)

Class II device (double insulation)

Name and address of the manufacturer

Read the user manual

For home use only

**REF**

Reference number

**SN**

Serial number

Keep dry

**IP21**

Protected against solid objects over 12.5 mm and vertically falling drops of water

Hazardous waste

Warning

Caution

DO

DON'T

ENU

### General use

**Use only as directed. Misuse can cause injury or death.**

**The device should only be used by adults.**

**Do not allow children to play with the device, it is NOT A TOY. (The power cord can easily wind around a child's neck.)**

**Do not leave the device unattended near children.**

**Do not use the device in the bathroom or in any other humid or wet environment.**

**Do not handle the power cord with wet hands.**

**Do not cover the vents on the back of the device while in use. Hot air exits the device through the vents.**

**The device is intended to be used only for removing nasal mucus from a child's nose.**

**Please consult your healthcare provider about when and how often you should clear your baby's nose or if redness occurs.**

**Keep out of reach of children.**

**Make sure that all parts are properly attached before using the device.**

**Make sure that the housing, the connecting tube and the Colibri head are not damaged or cracked.**

**Double-check that you have inserted the collared tube into the Colibri head after cleaning.**

## Cleaning, Maintenance and Storage

**Protect the power cord against damage.**

**Keep the power cord away from heat (e.g. furnace, heater or convector) because it can cause damage to the insulation.**

**Turn the device off and unplug the unit from the electrical outlet when assembling, disassembling and cleaning the device.**

**Don't break the power cord, don't place heavy items on the cord, to avoid damage. A damaged cord may cause electric shock.**

**Don't remove the mounting screws from the housing. Don't reconstruct the device for other purposes.**

**Don't leave the parts of the dismantled Colibri head within reach of children. Children may swallow the small parts.**

**Don't wash the housing or the power cord in water or any other liquid. Do not handle power cord with wet hands. This may cause electric shock or malfunction.**

**Make sure that all parts are dried thoroughly before the next use.**

**Double-check that you have inserted the collared tube into the Colibri head after cleaning.**

**Always unplug the device from the outlet when not in use.**

**To prevent damage, the tube connector should not be exposed to heat.**

**Store in a cool, dry place. Avoid direct sunlight or heat.**

## Power Source / Electrical Power

**WARNING**

**Protect the power cord against damage.**

**Switch off the device and unplug the unit from the electrical outlet when assembling, disassembling and cleaning the device.**

**In case of unexpected failure or malfunction, immediately switch off the device and unplug it from the electrical outlet.**

**Be sure to use the device only with a 110V AC.**

**Do not use the device if there is a breakage or crack in the housing or power cord.**

**Do not break the power cord, and do not place heavy items on it, because a damaged cord may cause electric shock.**

**CAUTION**

**Always unplug the device from the outlet when not in use.**

**Do not unplug the device from the electrical outlet until it cycles off completely.**

## 3. Intended use

The Nosiboo Pro Nasal Aspirator is intended for removing nasal mucus from a child's nose.

The Colibri head has a patented design that prevents mucus from entering the device.

### Intended purpose:

- \* to ensure clearer nasal breathing for infants
- \* to relieve symptoms of common childhood colds by aspirating nasal mucus
- \* to relieve childhood allergy-like symptoms by removing nasal mucus

### Do not use:

- \* if the child has hemophilia
- \* if the child has frequent nosebleeds
- \* if the child is taking blood-thinning medicines
- \* in the event of congenital malformations (e.g. cleft palate)

ENU

**WARNING**

**The device should only be used by adults.**


**CAUTION**

**The device is intended to be used only for removing nasal mucus from a child's nose.**


## 4. Before use

### 1. Assemble the Colibri head.

a) Clean and dry the parts of the Colibri head.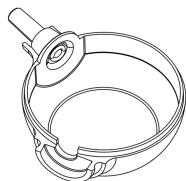

Colibri housing with connector

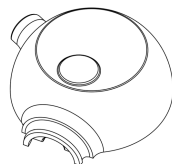
Colibri housing without connector


nose tip


collared tube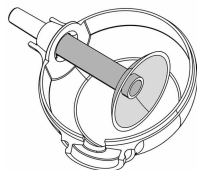


b) Insert the collared tube into the side of the Colibri housing that is supplied with a connector.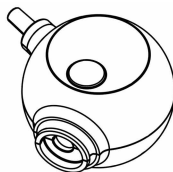

c) Place the Colibri housing that has no connector in the correct position.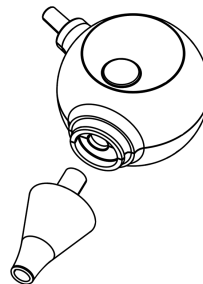


d) Push the nose tip into place.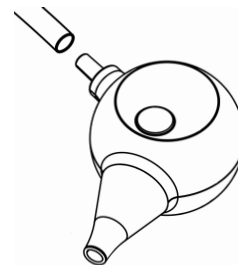


e) Attach one end of the connecting tube to the Colibri head.

## 2. Attach the Colibri head to the device.


- a) Attach the free end of the connecting tube to the device housing.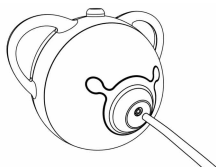

- b) Attach one end of the power cord to the device housing.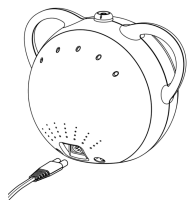

3. Plug the other end of the power cord into a 110V AC outlet.

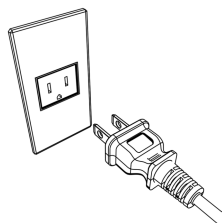4. The device is ready for use.

WARNING

**Be sure to use the device only with a 110V AC.  
Failure to do so may cause electric shock or fire.**

CAUTION

**Clean the parts of the Colibri head before use.**

**Make sure that all parts are dried thoroughly  
before use.**

**Do not use the Colibri head and device without  
the collared tube.**

ENU

## 5. Instructions for use

### 1. Select the correct suction power.

low setting for newborns

medium setting for babies

high setting for small children and for preschoolers

TIP: If the mucus is thin, you can use the device at a lower setting.  
If it is thick you can use the device at a higher setting than normal.

### 2. Hold your child in a stable position.

Sit your child onto your lap, hold them in your arms or sit them into an upright position supporting their back with a pillow. You may also sit them in an infant bouncer or infant seat.

### 3. Insert the clean, dry nose tip into the child's nostril.

The connection point of the connecting tube should point downwards while in use.

NOTE: Do not pinch or block the other nostril while using the nasal aspirator.

### 4. Turn on the device by pushing the control button.

NOTE: Any time you are not actively using the device, switch it off.  
After the motor cycles off completely, you can restart it.

The device turns itself off automatically after 15 minutes of continuous use but it can be restarted. In case of overuse or if insufficient ventilation is provided, the thermal switch switches off the device and will prevent reuse for approx. 45 minutes for safety reasons.

### 5. Move the nose tip around gently to make sure that you get all areas of the nostril.

TIP: The suggested length of contact aspiration is 10 seconds per nostril.

TIP: Dry, very thick mucus can be dissolved by a mild saline solution. After using the solution, wait a few minutes, wipe excess discharge with a tissue, then reuse the nasal aspirator as directed.

NOTE: To avoid damage to the device, do not use more than a teaspoon of solution at a time. The patented design of the Colibri head can hold up to a teaspoon of thin liquid and prevent mucus from entering the connecting tube.

**6. Switch off the device after use with the control button.**

**NOTE:** Always switch off the device after use.

The motor will run for a short time to cool down after you have switched off the device.

**TIP:** The Nosiboo Pro Nasal Aspirator can be used several times a day. It is recommended that you clear the airways before bed time and feeding to ensure the child breathes peacefully.

**NOTE:** Nose drops and other preparations should be used as directed by your healthcare professional.

ENU

**The device should only be used by adults.**

**Don't allow children to play with the device as it is NOT A TOY.**

**Don't cover the vents on the back of the device while in use. Hot air may exit the device through the vents.**

**Don't use the device in the bathroom or in any other humid or wet environment.**

**Don't handle the power cord with wet hands.**

**Make sure that all parts are dried thoroughly before use.**

**Always switch off the device after use with the control button.**


**Don't unplug the device from the electrical outlet until it is completely shut down.**


**Don't use the Colibri head and the device without the collared tube.**

## 6. Cleaning


1. Remove the connecting tube and the nose tip, then take apart the Colibri housing.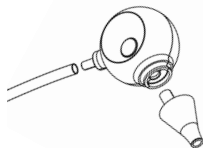


2. Wash the collared tube, the nose tip, and the Colibri housing in lukewarm water and dishwashing detergent and let them dry.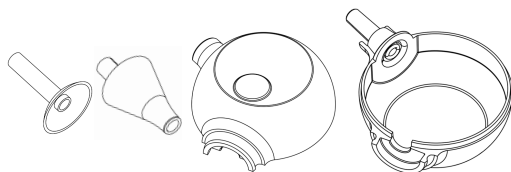

3. Clean the housing with a dry, soft cloth; do not use any liquids, detergents or chemicals.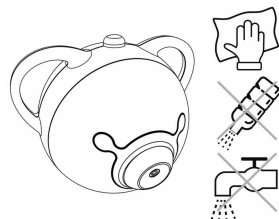

4. Wipe the connecting tube with a cloth if necessary.

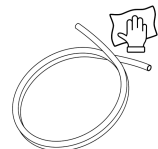**Switch off the device and unplug the unit from the electrical outlet when assembling, disassembling and cleaning its parts.**

**Don't wash the housing or the power cord in water or any other liquid.**

**Don't handle the power cord with wet hands.**

**Don't wash the connecting tube.**

**Thoroughly clean the Colibri head after use.**

**Make sure that all parts are dried thoroughly before the next use.**

**If liquid gets into the connecting tube due to improper use, make sure that it is completely dry before using it again.**

**Do not disassemble the housing.**

**To avoid damage, make sure the tube connector is not exposed to heat.**


## 7. Storage and maintenance

### Storage

1. Unplug the device after it has completely shut off and the cooling phase is over.

NOTE: After the cooling process has stopped, the power cord should be unplugged from the electrical outlet and from the device.

2. Disconnect the connecting tube from the device and the Colibri head.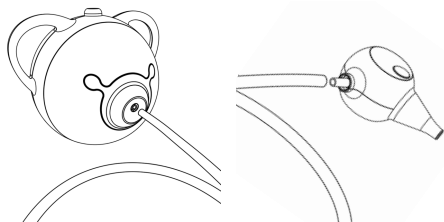


3. Keep the device in an enclosed space where it is protected from dust and damage (e.g. in a cabinet).

NOTE: Do not store near a heat source.

### Maintenance

There is a special air mesh behind the tube connector which prevents solid objects and dust from entering the housing. In the event of an unexpected drop in performance, it is recommended that you check the air mesh before the end of the cycle. If the air mesh is dirty or clogged, replace it with a spare air mesh. (A list of spare parts can be found in the "Accessories, spare parts" section.)

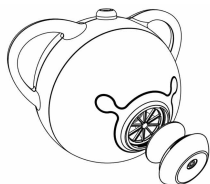**Protect the power cord against damage.**

**Keep the power cord away from heat.**

**Don't leave the parts of the dismantled Colibri head within the reach of children.**

**Don't break the power cord, and do not place heavy items on it, because a damaged cord may cause electric shock.**

**Make sure that all parts are dried thoroughly before assembly.**

**Store the device in a clean, dry place. Avoid direct sunlight or heat.**

ENU

## 8. Troubleshooting

| <b>Malfunction</b>                            | <b>Reason</b>                                                            | <b>Recommendation</b>                                                                                                                                                                                                    |
|-----------------------------------------------|--------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The device cannot be switched on.             | The plug is not connected.                                               | Connect the power cord to the device, then to the 110 V socket.                                                                                                                                                          |
| The device cannot be switched on.             | The device automatically switched off for safety reasons due to overuse. | For the sake of your child's health, stop using the nasal aspirator. The device can be used again in approx. 45 minutes.                                                                                                 |
| The suction intensity is decreased.           | The Colibri head is blocked or was incorrectly assembled.                | Take the Colibri head apart and clean it. After drying the parts of the Colibri head, assemble them according to the user manual.                                                                                        |
| The suction intensity is decreased.           | The air mesh is clogged.                                                 | Replace the air mesh according to the user manual's "Cleaning, maintenance and storage" section.                                                                                                                         |
| Liquid or mucus got into the connecting tube. | The Colibri head was not held in the correct position.                   | Switch off the device and clean and dry the Colibri head and the connecting tube. Check the air mesh and replace it if necessary.                                                                                        |
| Liquid or mucus got into the connecting tube. | The connection point of the tube is not pointed downwards.               | Switch off the device and clean and dry the Colibri head and the connecting tube. Check the air mesh and replace it if necessary.                                                                                        |
| Liquid or mucus got into the connecting tube. | Too much saline solution was used.                                       | Reduce the volume of the saline solution to less than a teaspoon (0.17 oz, 5 mL). Switch off the device and clean it, then dry the Colibri head and the connecting tube. Check the air mesh and replace it if necessary. |

## 9. Important information about electromagnetic interferences

Home wireless network equipment, mobile phones and wireless phones may affect the operation of certain medical equipment, as electromagnetic interference can occur between devices. Electromagnetic interference may cause harmful interference in the operation of certain medical devices, potentially causing emergency situations.

Due to the above mentioned reasons, medical devices must not interfere with any other equipment.  
Do not use near high-performance radiation-emitting devices.

The Nosiboo Pro Nasal Aspirator is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled.

The Nosiboo Pro Nasal Aspirator has been tested and found to comply with the requirements of EN 60601-1-2:2007 and of FCC Part 15, 2007.

## Disturbance emission tests

| Description of the tests            | Limits and test levels of the related Standard       | Result        |
|-------------------------------------|------------------------------------------------------|---------------|
| Radiated RF emission test           | <b>FCC Part 15, 2007</b> 30-3000 MHz class B         | <b>Passed</b> |
| Conducted disturbance voltage on AC | <b>FCC Part 15, 2007</b> 0.15-30 MHz class B         | <b>Passed</b> |
| Flicker                             | <b>EN 60601-1-2:2014</b><br><b>EN 61000-3-3:2009</b> | <b>NA</b>     |

## Immunity tests

| Description of the tests                                                  | Limits and test levels of the related Standard                                 | Result                                                                             |
|---------------------------------------------------------------------------|--------------------------------------------------------------------------------|------------------------------------------------------------------------------------|
| Immunity against radiated RF disturbances                                 | <b>EN 60601-1-2:2014</b><br>3V/m 0.08-2.5GHz<br>Modulation: 1 kHz, 80 % AM     | <b>Passed</b><br><b>Evaluation:</b> operation in compliance with the specification |
| Immunity against conducted RF disturbances on AC power and patient cables | <b>EN 60601-1-2:2014</b><br>3 Veff 0.15 – 80 MHz<br>Modulation: 1 kHz, 80 % AM | <b>Passed</b><br><b>Evaluation:</b> operation in compliance with the specification |
| Immunity against EFT signals                                              | <b>EN 60601-1-2:2014</b><br>±2 kV between L, N and ref. ground                 | <b>Passed</b><br><b>Evaluation:</b> operation in compliance with the specification |
| Immunity against surge voltages                                           | <b>EN 60601-1-2:2014</b><br>±1 kV between L and N                              | <b>Passed</b><br><b>Evaluation:</b> operation in compliance with the specification |
| Immunity against voltage dips                                             | <b>EN 60601-1-2:2014</b><br>100 %, 60 %, 30 %<br>0.5, 5, 25 periods            | <b>Passed</b><br><b>Evaluation:</b> operation in compliance with the specification |
| Immunity against short voltage interruptions                              | <b>EN 60601-1-2:2014</b><br>100 % for 5 periods                                | <b>Passed</b><br><b>Evaluation:</b> operation in compliance with the specification |
| Immunity against electrostatic discharges (ESD)                           | <b>EN 60601-1-2:2014</b><br>±8 kV air, ±6 kV contact                           | <b>Passed</b><br><b>Evaluation:</b> operation in compliance with the specification |

Flicker measurement was omitted because no flicker was observed.

ENU

## 10. Environment

Dispose of the device in accordance with your local recycling and waste disposal practices and requirements.

The Colibri head does not present a risk to the environment and can be disposed of as household waste.

## 11. Specifications

**Adjustable motor performance:** 210–380 W

**Differential pressure:** 0.03–0.08 bar

**Max. air flow:**

17,44 gpm (66 L/min) measured on the Nosiboo Pro housing

7,93 gpm (30 L/min) measured on the Colibri head

**Power supply:** 110 V 60 Hz

**IP class:** IP21

**Protection class:** II

**Type of applied parts:** BF

**Applied parts:** the complete housing

**Operation type:** no continuous operation

**Operation length:** on 13 min/approx. off 45 min

**Duration of continuous use:** max. 13 min

**Tripping temperature of the thermal switch:** 203°F (95°C)

**Average housing temperature:** 86–95°F (30–35°C) after 1 min of continuous use

**Noise level:** 55–62 dB

**Temperature range:** 14°F–104°F (-10°C–+40°C)

**Relative humidity:** max. 75%

**The expected life of the housing (electrical parts):** 3 years

**Expected life of the Colibri head:** 3 months

**Diameter of the nasal insertion end of the Colibri head:**

0.22 Inches (5.6mm)

**Length of the nasal insertion end of the Colibri head:**

min. 1.26 Inches (32mm)

**Weight (housing):** 2.26 lbs (1027g)

**Dimensions (housing, W x H x L):** 7.87x6.102x6.890 Inches (200x155x175 mm)

**Dimensions (complete package):** 7.441x7.441x 7.480 Inches (189x189x190 mm)

**We reserve the right to make technical changes.**

## 12. Accessories, spare parts

AC-05-03

Colibri head

AC-03-03

Connecting tube

AC-04-01

Air mesh

AC-01-01

Power cord

CO-01-03

Colibri head + connecting tube

AC-06-03

Tube connector

Parts can be ordered from the **customer service:**

Phone: 1-312-957-6046

E-mail: [customerservice@nosiboo.com](mailto:customerservice@nosiboo.com)

## 13. Limited warranty

### What is the period of coverage?

The limited warranty and any implied warranties, including implied warranties of merchantability and fitness for a particular purpose are valid for 12 months after the purchase date.

#### What is covered?

The warranty is only valid if the device was used according to its intended purpose and does not bear signs of damage detailed in the "What is not covered" section.

The warranty covers malfunctions that are caused by manufacturing defects (problems that were present at the time the device was sold).

#### Issue

**You experience "unusual" issues while operating the device, e.g. smoke emission, strong smell, the device buzzes or clicks unusually loudly, shakes, shorts out, cannot be adjusted via the control button, or does not turn off after 15 minutes.**

#### Is the warranty valid?

If you experience "unusual" issues while operating the device and the device does not bear traces of physical damage included in "What is not covered," the customer's device is covered by this warranty.

#### What is not covered?

The device is meant for personal use only. Any problem that is caused by overuse, abuse, misuse, or an act of God are not covered. Also, consequential and incidental damages are not recoverable under this warranty.

#### Issue

#### Is the warranty valid?

Signs of damage to the label indicate the device housing has been opened.

Due to misuse of the device, the warranty is not valid.

Signs of damage to the device housing indicate the device housing has been opened.

Due to misuse of the device, the warranty is not valid.

Excessive contamination of the air mesh, dirt in the device indicate the device has been misused.

Due to misuse of the device, the warranty is not valid.

Contamination in the connecting tube indicates the device has been misused.

Due to misuse of the device, the warranty is not valid.

The control button on top of the device is broken. This malfunction indicates external impact. Turning on and operating the device is life-threatening!

Due to abuse of the device, the warranty is not valid.

ENU

### **What will we do?**

The manufacturer will repair or replace the malfunctioning device if the device was used according to its intended purpose and if the malfunction occurs due to manufacturing errors (problems that were present at the time the device was sold).

### **What do you have to do?**

You must present a copy of your receipt/invoice or warranty certificate as proof of the date of sale.

### **How can you get service?**

For the nearest service location, please send an email to [customerservice@nosiboo.com](mailto:customerservice@nosiboo.com) or call us at 1-312-957-6046.

Send the malfunctioning device to the address we provide and we will take the necessary action to correct problems covered by this warranty.

### **Who pays what?**

You send the malfunctioning device to the service address at your expense and we will send the repaired device or a replacement back to you at our expense.

If the malfunction is covered by this warranty, we will repair or replace the device (depending on the malfunction and at our own discretion) free of charge.

If the malfunction is not covered by this warranty, we will contact you after the inspection and inform you of the repair cost so that you can decide how we should proceed.

Inspection is free of charge.

### **How does state law apply?**

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

Some states do not allow the exclusion or limitation of special, incidental or consequential damages; therefore, the above limitation or exclusion may not apply to you.

### **Return policy**

Nosiboo Pro Nasal Aspirator is a medical device. You may only return the device for a refund if the device is unused and undamaged and the protective foil on the Colibri head and on the device are intact.

If you have used the device or if the protective foils are not intact, please refer to the "Limited warranty" section above.

## International warranty certificate

ENU

Name and address of distributor:

Serial number of the Nosiboo Pro Nasal Aspirator:  
(found on the bottom of the device, on the label)

Purchase date:

Type of malfunction:

Date of malfunction:

Countersignature of distributor:  
(date, signature and seal)

Name and address of distributor:


Serial number of the Nosiboo Pro Nasal Aspirator:  
(found on the bottom of the device, on the label)

Purchase date:

Type of malfunction:

Date of malfunction:

Countersignature of distributor:  
(date, signature and seal)

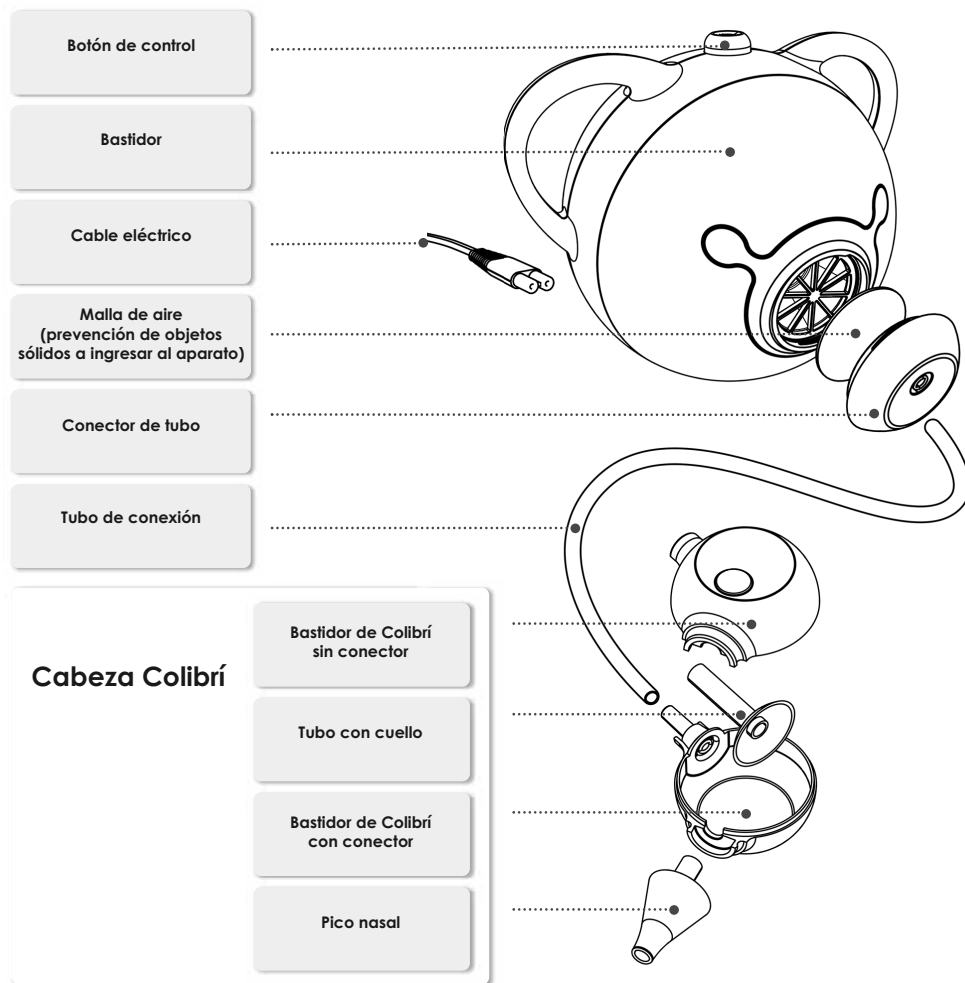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