# nosiboo<sup>\*</sup>pro<sup>®</sup>

electric nasal aspirator

## Safety instructions

EN

ES

FR

reddot award 2014  
winner

# Table of contents

**EN**

Safety instructions

**ES**

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**FR**

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# \* nosiboo<sup>®</sup> pro

electric nasal aspirator

## Safety instructions

Dear Customer,

thank you for purchasing our Nosiboo Pro Electric Nasal Aspirator.

Please, follow the instructions provided for your device. To obtain the very best performance for your nasal aspirator, please read the **Quick Guide** too. For more information we recommend reading the **User Manual**:

ATTRACT Kft. is an ISO 13485 certified medical device manufacturer. This product is an MDD Legacy Device.

Actor EUDAMED No.: HU-MF-000004252

Please keep these documents for future reference.

Please read all instructions carefully before using the device.

The warranty will be only validated if the device was used according to its intended purpose and does not bear signs of damage.

Nosiboo Pro is a hygienic medical device. You may only exercise your right of withdrawal if the device is undamaged and the protective foil on the device and the nose tip is intact. For hygienic reasons, we are unable to refund the product if the protective foil has been opened.

## Safety instructions

Type BF applied parts  
(increased protection against electric shock)

Double insulation

Class IIa medical device

Name and address of the manufacturer

Read the Safety instructions

Read the online User manual

For home use only

GTIN – Global Trade Item Number (UDI-DI)

Serial number (Unique device identifier)

Item number (type number)

Lot number (production identifier)

Keep dry

Protection against solid particles of diameter greater than 12.5 mm (0.492 inch); protected from condensation.

Hazardous waste

Caution!

EN

## General use

The Nosiboo Pro nasal aspirator is intended for removing nasal mucus from a child's nose.

The device is designed for use on the North American power grid. If used on any other electricity grid, the device will become defective and will not operate.

The device cannot be used in oxygen-rich environments and in clinical circumstances! The medical device is for home use only.

Use only as directed. Misuse can be dangerous.

The device should only be used by adults.

Do not use the nasal aspirator for nasal washing! Aspirating water with the electric device will result in loss of warranty.

In case of unexpected failure or malfunction, immediately switch off the device.

Do not allow children to play with the device, it is NOT A TOY. (E.g., the power cable can wind easily around a child's neck.)

Do not leave the device unattended near children.

Do not use the device in the bathroom or in any other humid or wet environment.

Do not handle the power cable and the metal parts of the device with wet hands. Do not cover the vents on the back of the device while in use.

The device is intended to be used only for removing nasal mucus from a child's nose. It shall not be used for any other purposes.

Keep the device and its accessories out of reach of children.

Make sure that all parts are properly attached before using the device.

Do not use the device without the air mesh.

Make sure that the housing, the connecting tube, and the Colibri head are not damaged or cracked.

Use the device only with the Colibri head supplied by the manufacturer. Use original Nosiboo parts! Our nasal aspirators work effectively only with original Nosiboo parts/accessories.

The part of the Colibri head attached to the connecting tube shall point downwards while in use.

Do not turn the Colibri head upside down meanwhile in use.

## Cleaning, maintenance and storage

Keep the device away from heat (e.g. furnace, heater, or convector) because it can cause damage to the insulation.

Do not tie knots on the power cable or the connecting tube.

To avoid damage, do not place heavy items on the cable. A damaged power cable may cause electric shock.

Do not use the power cable and the device if they are damaged or cracked. Do not remove the mounting screws from the housing. Do not reconstruct the device for other purposes. It may be life-threatening.

When you disassemble the Colibri head for cleaning, do not leave the parts within reach of children. Children may chew or swallow the small parts.

It is not allowed to use a dishwasher, boiling water, microwave oven, vaporizer, or baby bottle sterilizer for cleaning the spare parts, because they might deform by heat!

Do not wash the housing or the power cable with water or any other liquid. Do not handle the device with wet hands. This may cause electric shock or malfunction.

Always unplug the power cable from the outlet after using the device.

The device should be stored in a cool, dry place. Avoid direct sunlight or heat.

## Power source

Use the device only with the power cable supplied by the manufacturer. Failure to do so may result in fire hazard.

Be sure the plug of the power cable is compatible with your power outlet!

The device should be connected to or disconnected from the power network only if the control button is in OFF position (the device is switched off).

## Specifications

**Max. motor performance:** 500 W

**Adjustable motor performance:** 150-390 W

**Performance:** 420 VA

**Differential pressure:** 0.04-0.08 bar

**Max. air flow:** 30-42 l/min

**Power supply:** 110 V 60 Hz

**IP class:** IP21

**Protection class:** II

**Type of applied parts:** BF

**Applied parts:** the complete housing

**Operation type:** no continuous operation

**Operation length:** on 15 min/appr. off 50 min

**Duration of continuous use:** max 15 min.

**Safety switch release time:** after 15 minutes

**Tripping temperature of the thermal switch:** 95 °C

**Average housing temperature:** 30-35 °C after 1 min continuous use

**Noise level:** 60-70 dB

**Expected life of the housing (electrical parts):** 3 years

**Expected life of the Colibri head:** 3 months

**Diameter of the nasal insertion end of the standard nose tip:**

max. 7.0 mm

**Length of the nasal insertion end of the standard nose tip:** max.

8.0 mm

**Diameter of the nasal insertion end of the newborn's nose tip:**

max. 4.9 mm

**Length of the nasal insertion end of the newborn's nose tip:**

max. 12.6 mm

**Weight:** 1090 g

**Dimensions (housing, W x H x D):** 198 x 162 x 172 mm

**Dimensions (complete package):** 180 x 180 x 225 mm

**Operating conditions:**

1. Temperature range: -5 °C to +40 °C

2. Relative humidity: max. 75%

3. Atmospheric pressure / barometric pressure range: 525-795 mmHg / 70-106 kPa

**Transportation conditions:**

1. Temperature range: -20 °C to +65 °C

2. Relative humidity: 10-75%

3. Atmospheric pressure / barometric pressure range: 150-795 mmHg / 20-106 kPa

**Storage conditions:**

1. Temperature range: -20 °C to +65 °C

2. Relative humidity: 10-75%

3. Atmospheric pressure / barometric pressure range: 525-795 mmHg / 70-106 kPa

**MD Category:** II.a

**NBOG:** MD1102

**MDD/MDR:** MDD Legacy Device

**Actor EUDAMED No.:** HU-MF-000004252

We reserve the right to make technical changes.

The manufacturer shall not be held responsible for the failure of products from unauthorised import sources!

## Contact

Manufacturer contact details:

ATTRACT Kft.

H-7622 Pécs, Siklósi út 1/1., Hungary

Tel.: (312) 957-6046 (Mo-Fr: 8 am – 4 pm CET)

E-mail: customerservice@nosiboousa.com

PRRC (Person Responsible for Regulatory Compliance):

ATTRACT Kft. – Quality Manager

H-7622 Pécs, Siklósi út 1/1., Hungary

Tel.: (312) 957-6046 (Mo-Fr: 8 am – 4 pm CET)

E-mail: quality@nosiboo.com

For more information we suggest reading the user manual and our quick guide as well.

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